



AP-QUATBLEND 8

DESCRIPTION

AP-QUATBLEND 8 is a concentrated, neutral disinfectant/cleaner/deodorizer, biodegradable product made for use on surgical and medical surfaces, general healthcare industry. It requires no additional mixing, only water, and is effective against a broad range of bacteria, including Covid-19, as well as germs and other pathogenic micro-organisms.

USES OF THE PRODUCT

AP-QUATBLEND 8 is a cleaner and disinfectant, for use in all areas in hospitals, medical practices and medical baths. Usable for cleaning and disinfection of protective masks and disinfection of treatment surfaces. The product is also suitable in food sector (e.g. hospital or restaurant kitchen) and is effective against: Bacteria, yeasts, Enveloped viruses (incl. HBV, HCV, HIV) and Polyomavirus · Aldehyde-free · Perfume-free. The product is also suited for rapid disinfection (5-10 minutes).

PRECAUTIONS AND RECOMMENDATIONS

- AP-QUATBLEND 8 is safe to use and cleans and disinfects a variety of fixtures and surfaces.
- The product is for normal use (not delicate); however, it is recommended to use plastic gloves and glasses for prolonged handling.
- Do not ingest, if swallowed do not induce vomiting, seek medical attention.
- Keep out of reach of children and pets.
- The product leaves no residue, leaving floors and surfaces looking and feeling clean and fresh.
- Avoid prolonged direct contact with the skin and especially with the eyes.
- In case of contact with eyes and/or skin, wash with plenty of water for at least 15 minutes. If eye irritation persists, consult a doctor.
- Use in ventilated areas.
- AP-QUATBLEND 8 is not considered toxic, ONLY, by ingestion and does not contain known carcinogens.

• USAGE INSTRUCTIONS

- The product is concentrated for use in health care environments; on medical devices, for the prevention of hospital cross-infections in all hospital areas and medical practices.
- Not only is it suitable for spraying to do surface cleaning, but it is also effective for mopping and for surface cleaning with a rag.
- Dilute in plain water to a maximum of 50% to maintain required medical standard and not affect the efficiency of the diluted solution.

INGREDIENTS

Chemical:Cas No%

- Isopropyl Alcohol67-63-012
- Bio-Surfactants Blend25154-54-58
- Quaternary Blend8001-54-55
- InertN/A77

PHYSICAL AND CHEMICAL DETAILS

- **Appearance and smell** _____ *non-viscous liquid with alcohol odor, blue color*
- **Boiling point** _____ *> 82 °C*
- **Solubility in water** _____ *total soluble*
- **pH** _____ *7 – 8*
- **Density** _____ *0.79 – 0.99 g/mL*
- **Slightly volatile**
- **Flash point** _____ *19°C*

SAFETY INFORMATION

- Review the safety data sheet (SDS) before using this product.
- Arupro Chemicals assumes no responsibility for injuries, damages or losses resulting from improper use of the product.
- Issue date: August 2026. Reviewed by Rómulo Hernández, Bachelor of Chemical Engineering, MBA, Tulane University
- Prepared by: Arupro Chemicals Technical Department
- NOTE: This information is based on research results and development and can be considered reliable.



PRODUCT PRESENTATION

- 1 Gallon Bottle